Occlutech Delivery Set Product specifications



Art. No.	Size [F]	Length Sheath [mm]	Length Loader [mm]	Dilator Tip [F]	Hemo-Valve [F]	Distal Angle
45° Length 800 mm						
51DS007	7	800	120	25	70	45°
51DS008	8	800	160	25	70	45°
51DS009	9	800	160	25	70	45°
51DS010	10	800	160	25	70	45°
51DS011	11	800	200	25	70	45°
51DS012	12	800	200	25	70	45°
51DS014	14	800	200	25	70	45°
180° Length 800 mm						
51US006	6	800	120	25	70	180°
51US007	7	800	120	25	70	180°
51US008	8	800	160	25	70	180°
51US009	9	800	160	25	70	180°

Availability subject to regulatory approval. Product data may be subject to change. Contact your Occlutech representative for information.

Quality made in Europe

Occlutech adheres to the highest quality standards. We never compromise on quality. Our dedication to patients and the medical community is the main driver of our vision. We want to be at the forefront of developing technologies and products that matter, that make a difference and that help our customers to be at the leading edge of better treatment for structural heart disease.

Landskronavägen 2, SE-252 32 Helsingborg, Sweden





Occlutech Delivery Set





Perfect Delivery

Features & Benefits

A. Dilator

The dilator (A) is made from smooth conformable low friction radio-opaque material. The dilator is inserted into the sheath (B) to form a supporting inner body. The inner lumen of the dilator has 0.035" guide wire compatibility.

B. Delivery sheath


The sheath (B) consists of a smooth outer layer and a low friction inner layer reinforced with metal braiding. The braid improves the strength of the sheath and provides radiopacity. The sheath has a proximal female connector for connection to the dilator (A) or to the loader (C).


C. Transparent loader

The loader (C) is made of low friction material and is transparent to allow inspection for air bubbles after pulling the occluder into the loader.

D. Y-connector with hemostasis valve

The Y-connector (D) is a commercially available connector with a sideport and hemostasis valve. The valve allows for leak-proof passage of the Occluder delivery wire. The side-port permits flushing before and during implantation.

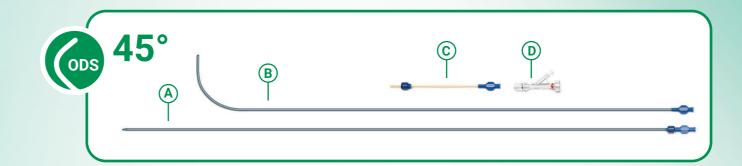

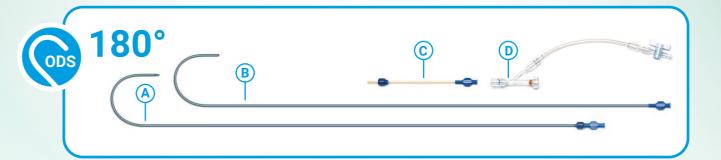

Occlutech Delivery Set







